

BC5083-25

DOLPHIN 32MM SLIDER BRACKET FOR SHOWER HEAD HOLDER

INSTALLATION AND OPERATING INSTRUCTIONS



September 2022

D
DOLPHIN
ELEMENTS

Available colours: **CHROME SATIN (CS)**

Congratulations on the purchase of your shower bracket. This shower bracket has been designed to provide users with an adjustable shower head height and to make it easier for users with limited dexterity.

Warnings:


- › On receiving delivery of this product, a thorough check must be carried out and any defects immediately reported. The product must be stored in a dry place and not stacked under other equipment.
- › Never alter the bracket components as this could affect the strength and integrity of the bracket.
- › Improper installation/use can cause accidents and injuries.
- › After installation a functional test must be carried out prior to use.
- › These instructions should be retained by the user of the product for its whole life.
- › This bracket is designed to hold a shower head and should not be used in a way that would apply the users` weight to the bracket.
- › Do not use the bracket for any purpose, other than, that for which it was designed.

The information in this document reflects products at the date of printing. Dolphin Solutions Ltd reserves the right, subject to all applicable laws, at any time, at its sole discretion, and without notice, to discontinue or change the features, designs, materials and other specifications of its products, and to either permanently or temporarily withdraw any of the foregoing from the market. All information in this document is provided "as is" without warranty of any kind, either expressed or implied, including but not limited to any implied warranties of merchantability, fitness for a particular purpose, or non-infringement. Dolphin Solutions Ltd assumes no responsibility for errors or omissions in the information presented in this document. In no event shall Dolphin Solutions Ltd be liable for any special, incidental, indirect or consequential damages of any kind, or any damages whatsoever arising out of or in connection with the use or performance of this information. The tradenames, trademarks, logos and service marks presented in this document, including their design, are the property of Dolphin Solutions Ltd or other third parties and you are not permitted to use them without the prior written consent of Dolphin Solutions Ltd or such third party as may own them.

INSTALLATION

Installation should be carried out by a suitably qualified tradesman.

1. Once the shower slider/grab rail has been installed this bracket can be installed on it.
2. Take the infill part out of the back of the bracket.
3. Push the bracket onto the shower slider/grab rail until it clamps.
4. Put the infill back into the back of the bracket.
5. Push the grey clip through the bracket and the infill to lock the infill onto the back of the bracket.
6. Ensure the grey clip is pushed all the way through and then secure the grey clip on the opposite side with the screw provided
7. Always carry out a functional test after installation and report any issues immediately.
8. Please leave these instructions with the user.

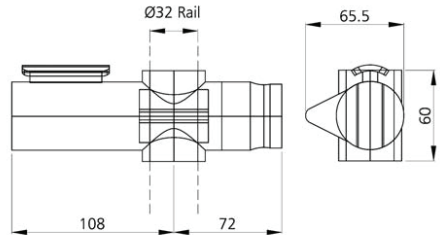

OPERATING INSTRUCTIONS

Using this product:

ALWAYS check the bracket is fitted securely prior to use. The bracket clamps the slider/grab rail, this can be released by squeezing the side of the bracket with the large button. To clamp the rail on the bracket again, release the button.

Cleaning and maintenance:

The rail must be checked regularly by qualified personnel to ensure that it is in perfect working order. This bracket can be cleaned using only a very mild detergent. Do not use acidic, alkaline, chlorine based or abrasive cleaning materials.

Disclaimer:

We accept no liability whatsoever for personal injury or damage to property that is or has been caused due to incorrect assembly, installation, or use of this product. Failure to install/use this product in line with these instructions.

Every effort has been made to ensure as much accuracy as possible. However, due to print restrictions exact colour match is not always possible. Featured products may also be changed from time to time, but are correct at time of print.



Dolphin Solutions Ltd

Southpoint, Compass Park, Bodiam,
Robertsbridge, TN32 5BS

Tel: +44 (0)1424 20 22 24

Fax: +44 (0)1424 205 200

info@dolphinsolutions.co.uk

www.dolphinsolutions.co.uk